
INgezim® Moquillo IC

R.15.CDV.K42

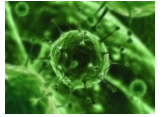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

INgezim® MOQUILLO IC

R.15.CDV.K42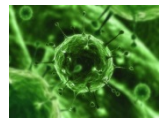

INgezim® MOQUILLO IC is a direct immunochromatographic assay, based on Lateral Flow technique, which uses 2 monoclonal antibodies (MAbs) specific of Canine Distemper virus (CDV).

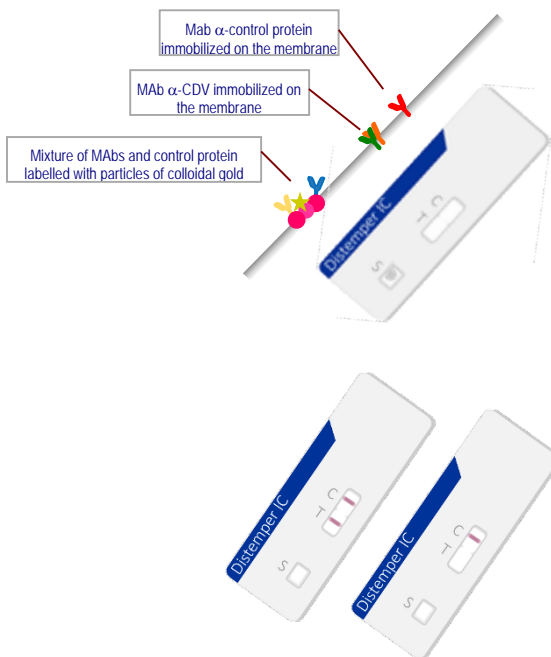TECHNICAL BASIS OF THE KIT

The device consists on a plastic casing with 2 windows:

- **Window for sample addition:** Contains a MAb specific CDV and a Control Protein. Both labelled with colloidal gold.
- **Window for results interpretation:** Contains a test line (T) with a CDV specific MAb and a control line (C) with a MAb specific to the control protein.

APPLICATION

Detection of Canine Distemper virus in biological samples (conjunctiva, tears, nasal discharges, etc) of dogs.

INTERPRETATION OF THE RESULTS

Positive sample: Pink line in the Test zone and a pink line in the Control zone.

Negative sample: Only a pink line in the Control zone.

Not valid assay: No pink line in the Control zone

SENSITIVITY AND SPECIFICITY

Correspondence with RT-PCR technique.

140 conjunctiva (ocular discharges) samples of dogs collected from the clinicians in Italy were analyzed. The results were compared with the obtained by RT-PCR. The assay showed 98.8% and 100% sensitivity and specificity respectively regarding RT-PCR with 99.3 accuracy and 100%-98.2% positive and negative predictive values respectively.

		RT-PCR		
		POS	NEG	TOT
Moquillo IC	POS	84	0	84
	NEG	1	55	56
	TOT	85	55	140

COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Dropper vial with dilution buffer.
- Pipettes

PRODUCT MANUFACTURED BY
AGROLABO AND DISTRIBUTED BY
INGENASA

SHELF LIFE: 18 months
Stored: 4°C-25°C

Ed.090418