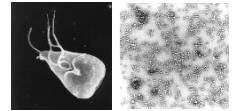
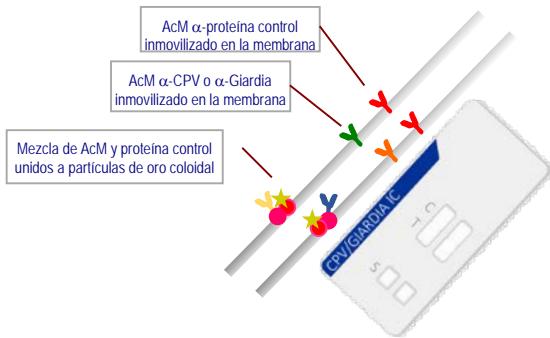
INgezim PARVO-GIARDIA IC

R.15.CGD.K42

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

INgezim PARVO-GIARDIA IC

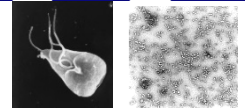R.15.CGD.K42

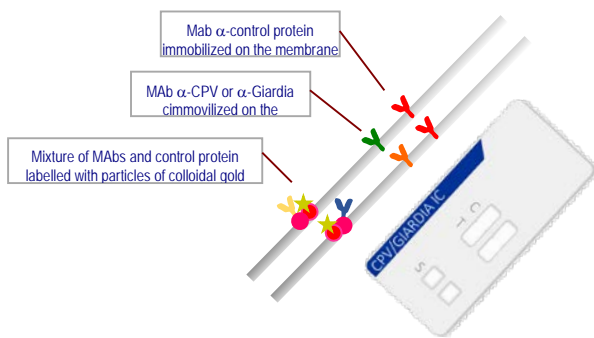INgezim PARVO-GIARDIA IC is a direct immunochromatographic assay, based on Lateral Flow technique, which uses 2 monoclonal antibodies (MAbs) specific of two different epitopes of Canine Parvovirus (CPV) and 2 MAbs specific of two different epitopes of Giardia lamblia.

TECHNICAL BASIS OF THE KIT

The device consists on a plastic casing with 4 windows:

- **2 Windows for sample addition:** One contains a MAb specific CPV and a Control Protein and the other one contains a MAb specific of Giardia lamblia and a Control Protein. All of them labelled with colloidal gold.
- **2 Windows for results interpretation:** One contains a test line (T) with a CPV specific MAb and a control line (C) with a MAb specific to the control protein. The other one contains a test line (T) with a Giardia lamblia specific MAb and a control line (C) with a MAb specific to the control protein

APPLICATION

Detection of Canine Parvovirus types 2, 2a, 2b and 2c and Giardia lamblia in biological samples (faeces and intestinal fluid) of dogs (CPV) and cats and dogs (Giardia lamblia).

INTERPRETATION OF THE RESULTS

Positive sample: Pink line in the Test zone and a pink line in the Control zone.

Negative sample: Only a pink line in the Control zone.

Not valid assay: No pink line in the Control zone

	CPV (+)	CPV (-)
Giardia (+)		
Giardia (-)		

SENSITIVITY AND SPECIFICITY (CPV)

Correspondence with hemagglutination inhibition test (IHA).

201 samples of dogs collected from the clinicians in Italy. Samples consisted on two different matrixes: serum for IHA and faeces for IC test. The assay showed 99.2% and 100% sensitivity and specificity respectively regarding IHA.

		IHA		
		POS	NEG	TOT
PARVO IC	POS	120	0	120
	NEG	1	80	81
	TOT	121	80	201

SENSITIVITY AND SPECIFICITY (G. lamblia)

Correspondence with a commercial kit.

120 feline and canine samples from different clinicians were analyzed. Results were compared with the ones obtained by a commercial kit. The assay shows 96% and 100% sensitivity and specificity respectively.

		Commercial Kit		
		POS	NEG	TOT
Giardia IC	POS	72	0	72
	NEG	3	45	48
	TOT	75	45	120

COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Diluent tubes to prepare the dilution of the faeces.

PRODUCT MANUFACTURED BY
AGROLABO AND DISTRIBUTED BY
INGENASA

SHELF LIFE: 18 months
Stored: 4°C-25°C

Ed.110418