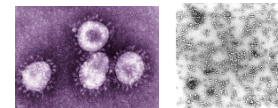
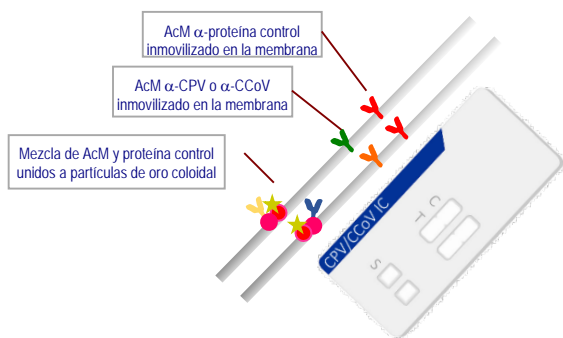
INgezim PARVO-CORONA IC

R.15.CCD.K42

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

INgezim PARVO-CORONA IC

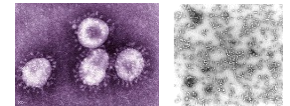R.15.CCD.K42

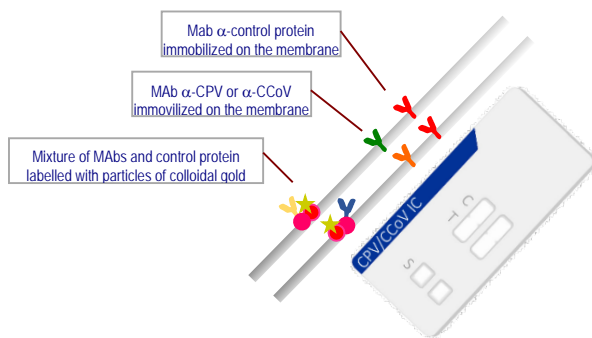INgezim PARVO-CORONA IC is a direct immunochromatographic assay, based on Lateral Flow technique, which uses 2 monoclonal antibodies (MAbs) specific to Canine Parvovirus (CPV) and 2 MAbs specific of Canine Coronavirus (CCoV).

TECHNICAL BASIS OF THE KIT

The device consists on a plastic casing with 4 windows:

- **2 Windows for sample addition:** One contains a mixture of MAbs specific CPV and a Control Protein and the other one contains a mixture of MAbs specific CCoV and a Control Protein. All of them labelled with colloidal gold.
- **2 Windows for results interpretation:** One contains a test line (T) with CPV specific MAbs and a control line (C) with a MAb specific to the control protein. The other one contains a test line (T) with CCoV specific MAbs and a control line (C) with a MAb specific to the control protein

APPLICATION

Detection of Canine Parvovirus types 2, 2a, 2b and 2c and Canine Coronavirus in biological samples (faeces) of dogs.

INTERPRETATION OF THE RESULTS

Positive sample: Pink line in the Test zone and a pink line in the Control zone.

Negative sample: Only a pink line in the Control zone.

Not valid assay: No pink line in the Control zone

	CPV (+)	CPV (-)
CCoV (+)		
CCoV (-)		

SENSITIVITY AND SPECIFICITY (CPV)

Correspondence with inhibition hemagglutination test (IHA).

201 samples of dogs collected from the clinicians in Italy. Samples consisted on two different matrixes: serum for IHA and faeces for IC test. The assay showed 99.2% and 100% sensitivity and specificity respectively regarding IHA.

		IHA		
		POS	NEG	TOT
PARVO IC	POS	120	0	120
	NEG	1	80	81
	TOT	121	80	201

SENSITIVITY AND SPECIFICITY (CCoV)

Correspondence with RT-PCR.

194 samples of dogs collected from the clinicians. The assay showed 95.2% and 99.2% sensitivity and specificity respectively regarding RT-PCR.

		RT-PCR		
		POS	NEG	TOT
CORONA IC	POS	60	1	61
	NEG	3	130	133
	TOT	63	131	194

COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Diluent tubes to prepare the dilution of the faeces
- Pipettes

PRODUCT MANUFACTURED BY
AGROLABO AND DISTRIBUTED BY
INGENASA

SHELF LIFE: **18 months**
Stored: 4°C-25°C

Ed.090418