
INgezim® HEARTWORM IC

R.15.HW.K42

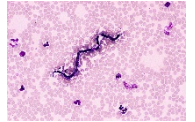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

INgezim® HEARTWORM IC

R.15.HW.K42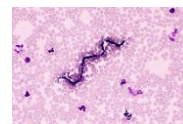

INgezim® HEARTWORM IC is a direct immunochromatographic assay, based on Lateral Flow technique, which uses monoclonal antibodies (MAbs) specific to *Dirofilaria immitis*.

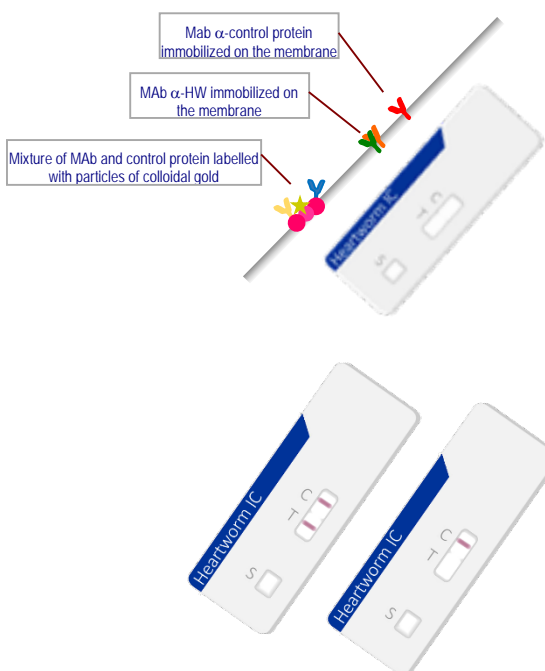TECHNICAL BASIS OF THE KIT

The device consists on a plastic casing with 2 windows:

- **Window for sample addition:** Contains a MAb specific of *Dirofilaria immitis* and a Control Protein labelled with colloidal gold.
- **Window for results interpretation:** Contains a test line (T) with a *D. immitis* specific MAb and a control line (C) with a MAb specific of the control protein.

APPLICATION

Detection of *Dirofilaria immitis* in serum, plasma and blood samples of dogs.

INTERPRETATION OF THE RESULTS

Positive sample: Pink line in the Test zone and a pink line in the Control zone.

Negative sample: Only a pink line in the Control zone.

Not valid assay: No pink line in the Control zone

SENSITIVITY AND SPECIFICITY (CPV)

Correspondence with ELISA test.

203 canine samples were analyzed in this study. The assay showed 100% and 99% sensitivity and specificity respectively, regarding ELISA. The accuracy was 99.5% with Positive and Negative predictive values 99% and 100% respectively.

		ELISA		
		POS	NEG	TOT
HEARTWORM IC	POS	105	1	106
	NEG	0	97	97
	TOT	105	98	203

COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Dilution buffer dropped bottle.
- Plastic pipettes

PRODUCT MANUFACTURED BY
AGROLABO AND DISTRIBUTED BY
INGENASA

SHELF LIFE: 18 months
Stored: 4°C-25°C

Ed.110418