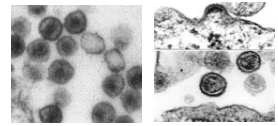
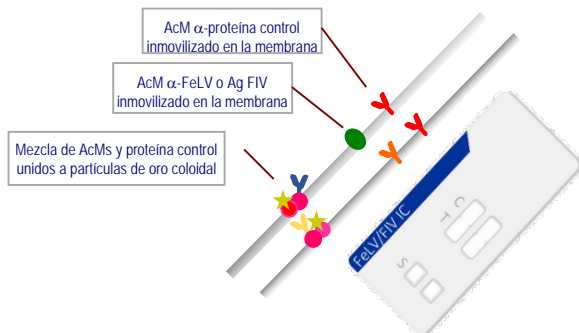
## INgezim® FeLV/FIV IC

R.16.FFD.K40

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

## INgezim® FeLV/FIV IC

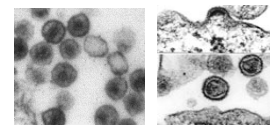R.16.FFD.K40

**INgezim® FeLV/FIV IC** is a immunochromatographic assay based on the lateral flow technique to detect Feline Leukemia Virus (FeLV) antigen and antibodies specific of Feline Immunodeficiency Virus (FIV). FeLV detection is based on a direct assay using monoclonal antibodies (MAbs) specific of 2 different epitopes in FeLV p27 protein. Detection of antibodies specific of FIV is based on an indirect assay using a specific FIV antigen (Ag) and a MAb specific of feline IgG.
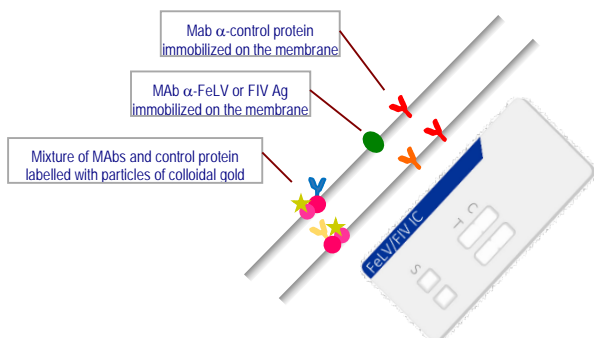
### TECHNICAL BASIS OF THE KIT

The device consists on a plastic casing with 4 windows:

- **Window for sample addition:**
  - a. **FeLV:** contains a MAb specific p27 of FeLV and a Control Protein. All of them labelled with colloidal gold.
  - b. **FIV:** contains a MAb specific of feline IgG and a Control Protein. All of them labelled with colloidal gold.
- **Window for results interpretation:**
  - a. **FeLV:** contains a test line (T) with a FeLV specific MAb and a control line (C) with a MAb specific to the control protein.
  - b. **FIV:** contains a test line (T) with FIV antigen and a control line (C) with a MAb specific to the control protein.

### APPLICATION

Detection of FeLV p27 protein and detection of antibodies specific of FIV. Samples used are serum, plasma and blood of cats.

### INTERPRETATION OF THE RESULTS

**Positive sample:** Pink line in the Test zone and a pink line in the Control zone.

**Negative sample:** Only a pink line in the Control zone.

**Not valid assay:** No pink line in the Control zone

	FeLV (+)	FeLV (-)
FIV (+)		
FIV (-)		

### SENSITIVITY AND SPECIFICITY (FeLV)

#### Correspondence with ELISA test.

160 serum and plasma samples from different clinicians were analyzed. Results were compared with the ones obtained by a commercial ELISA. The assay showed 100% and 95% sensitivity and specificity respectively.

FeLV IC	ELISA		
	POS	NEG	TOT
POS	35	6	41
NEG	0	119	119
TOT	35	125	160

### SENSITIVITY AND SPECIFICITY (FIV)

#### Correspondence with Western Blot (WB) test.

351 serum and blood samples from different clinicians were analyzed. Results were compared with the ones obtained by WB. The assay showed 98.5% and 99.7% sensitivity and specificity respectively.

FIV IC	WB		
	POS	NEG	TOT
POS	64	1	65
NEG	1	285	286
TOT	65	286	351

### COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Dropper vial with dilution buffer.
- Pipettes

PRODUCT MANUFACTURED BY  
AGROLABO AND DISTRIBUTED BY  
INGENASA

**SHELF LIFE: 18 months**  
Stored: 4°C-25°C

Ed.090418