Ingezim FeLV/FIV IC

Prod. Ref: **1.6.FFD.K40/12**

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INgezim FeLV/FIV IC

Prod. Ref: **1.6.FFD.K40/12**

Double immunocromatographic device for detection of FeLV antigen and specific antibodies to FIV in feline blood, serum or plasma samples.

INMUNOLOGIA Y GENETICA APLICADA, SA (INGENASA)

C/ Hnos Garcia Noblejas, 39 – 28037 MADRID (Spain)

Tel: (+34)91 3680501

www.ingenasa.com

Versión doc: 052016

KIT COMPONENTS:

Componente	Unidades
Immunocromatographic devices	12
Pasteur disposable pipettes	12
Dropper containing buffer	1

OTHER MATERIALS OR REAGENSTS NEEDED NOT PROVIDED:

– None

KIT CONSERVATION:

All components of the kit should be stored between + 4 ° C and + 25 ° C, in its original packaging until use.

PRINCIPLE OF THE TEST:

The test is based on the immunocromatografic technique. In the case of FeLV, it detects antigen prom the FeLV and in case of FIV, detects the presence of specific antibodies to this virus.

PRECAUTIONS:

- Carefully follow the instructions.
- Place all reagents to room temperature at least 20 min before use.
- Never open the foil pouch containing the device if not used immediately. It could be hydrated and become unusable. Open only immediately before use.
- Do not mix instructions or reagents from different kits.
- Avoid any contamination of reagents.
- Do not use the kit once exceeded their expiration date.
- Do not eat, drink or smoke when samples and / or reagents are handled.
- Do not touch the chromatographic membrane with any object, especially with the dropper tip.

INSTRUCTIONS:

A.- SAMPLE PREPARATION:

The sample of choice for diagnostic purposes, shall be blood, serum or plasma. You can use fresh samples immediately after the extraction and you can store samples up to 5 days after extraction in refrigeration conditions. If you plan to make the assay latter, it is better to store samples under frozen conditions.

B.- TEST PROCEDURE:

Remove the diagnostic device from its envelope and place it on a flat surface. Discard the bag of silica gel. NOTE: Only open the bag at the time of going to test.

Using the Pasteur pipette provided, drop 1 drop of sample on each window "S" of the device. (strip 1 is FeLV and strip 2 is FIV)

CAUTION: The pipette should be placed vertically on the window and keeping the tip 1 cm to avoid touching the membrane.

After the sample addition, drop on each "S" window" of the device 2 drops of buffer.

CAUTION: After adding the 2 drops of buffer in the "S" window of FeLV, check carefully how the migration front go forward. If after 2 minutes of migration the front do not reach the results window, add an additional drop of buffer.

C.- READING AND RESULTS INTERPRETATION:

Keep the device on a flat surface and untouched for 10 minutes, after which proceed to reading the result:

	FeLV (+)	FeLV (-)
FIV (+)		
FIV (-)		

It is advisable not to give any negative sample definitely until you have 10 minutes elapsed from the placement of the sample into the device.