INgezim® Brucella Porcina

R.11.BP.K1

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INgezim® Brucella Porcina

R.11.BP.K1

INgezim® Brucella Porcina is an immunoenzymatic assay based on indirect ELISA technique, which uses a monoclonal antibody (Mab) specific to porcine IgG immunoglobulins, and lipopolysaccharide (SLPS) from *smooth brucella*.

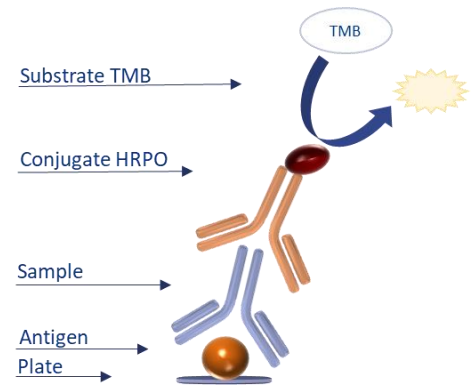
KIT FEATURES

APPLICATION

Detection of specific antibodies to LPS of smooth *Brucella* in porcine serum samples.

TECHNICAL BASE

- Plates are supplied coated with antigen (inactivated smooth *Brucella* antigen, LPS). Samples are added to the wells and incubated.
- If the samples contain specific antibodies to *Brucella*, they will bind to the antigen.
- When the conjugate (monoclonal antibody specific to porcine IgG, conjugated with peroxidase, AcM-PO) is added, it will bind to the antibodies bound to the antigen. This binding is revealed by colorimetric reaction after addition of the substrate.

RESULTS INTERPRETATION

The assay establishes a cut off, which will classify the samples as **Positive** or **Negative**, depending on the value of the optical density of the sample.

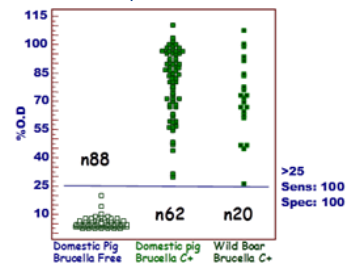
ASSAY VALIDATION

Use of OIE Reference Serum

In order to validate the assay respect to OIE regulation, the Reference Serum: "Suis International Standard Serum" (ISaBS) was used. The assay is able to detect it as positive at 1/16 dilution fulfilling the OIE requirements.

Diagnostic sensitivity and specificity

- Study 1.** (AFFSA) A panel of 998 positive and negative sera previously classified by Rose of Bengale (RBT) and the competition ELISA INgezim® Brucella Compac 2.0 was analysed. Results obtained indicated 92.38% correspondence with RBT and 97.8% with INgezim® Brucella Compac 2.0.
- Study 2.** (Unidad de Sanidad Animal. CITA. Zaragoza. España) The following gold standard sera were used: 88 domestic pigs from Brucella free herds, 62 domestic pigs naturally infected with *B. suis biovar 2*, and 20 wild boars naturally infected with *B. suis biovar 2* (these 82 positive sera were confirmed by individual bacteriology). The pig sera were analysed by RBT. Moreover, with the obtained results with these 3 groups, a ROC analysis was performed. **The assay showed 100% sensitivity and specificity.**
- Study 3.** (Unidad de Sanidad Animal. CITA. Zaragoza. España). 555 sera from herds showing different status concerning *B. suis* were analysed. Groups 1 and 2 suffered from reproductive problems, groups 3 and 4 did not suffer from reproductive problems.

	Group 1 (n=145, isolation positive)			Group 2 (n=134, farms showing many RBT reactors)			Group 3 (n=60, farms showing very few RBT reactors)			Group 4 (n=216, farms without RBT reactor)		
	RBT	CFT	INgezim BP	RBT	CFT	INgezim BP	RBT	CFT	INgezim BP	RBT	CFT	INgezim BP
POSITIVE	132 (91%) ¹	90 (62%)	120 (82,7%)	43 (30.1%)	19 (14.1%)	51 (38%)	3 (5%)	0 (0%)	6 (10%)	0 (0%)	0 (0%)	7 (3.2%)
NEGATIVE	13	48	25	91	89	83	57	60	54	216	205	209
PA ²		7			26			0			11	

¹ Apparent prevalence percentage in each test. ² number of sera showing anticomplementary reactions.

ATTENTION: cross-reaction in 2 of 66 sera of *Yersinia enterocolitica* infected animals has been detected.

Standardization of the new European Reference Serum for porcine Brucellosis (EUPBSS) using INgezim® Brucella Porcina

The assay has been used in a Ring Test organised by ANSES in 2014 (7th workshop of the EU Brucellosis NRLs. 2014) as a reference technique for the validation of the EUPBsSS OIE reference serum.

KIT COMPOSITION

- 96 well microtitration plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with Conjugate
- Bottles with Washing Solution
- Bottles with Diluent
- Bottles with substrate (TMB)
- Bottles with Stop Solution

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Eurofins-INGENASA, S.A.