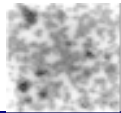
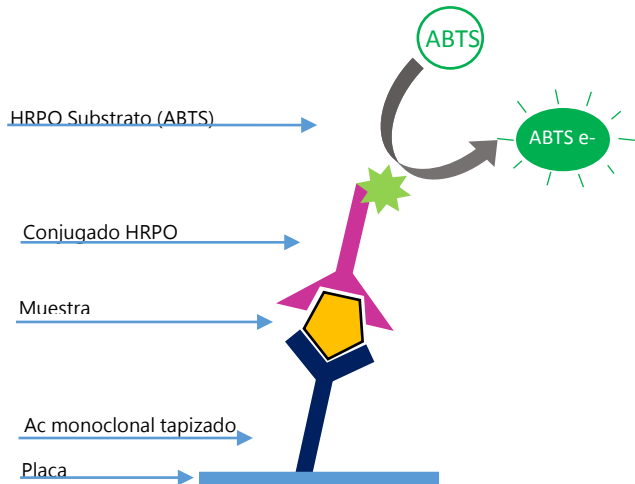
INgezim CPV DAS

R.15.CPV.K2

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
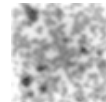
INgezim CPV DAS

R.15.CPV.K2

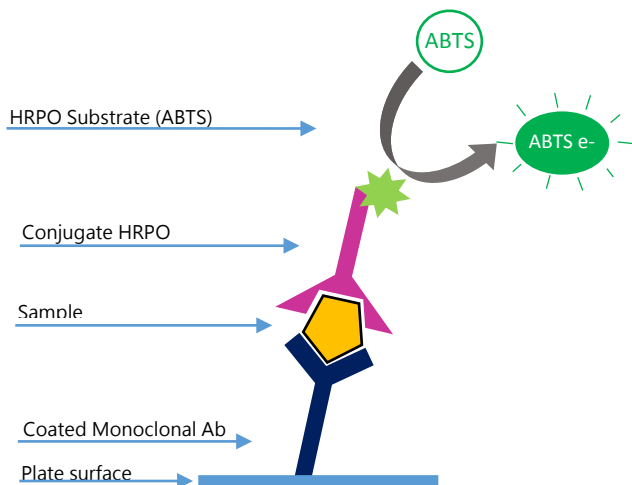INgezim CPV DAS is an enzymatic assay based on a Double Antibody Sandwich ELISA technique, which uses monoclonal antibodies (MAb) specific to Canine Parvovirus (CPV) VP2 protein.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with a monoclonal antibody specific of CPV protein vp2. Samples are added and incubated.
2. If samples contain CPV, they will bind to the specific monoclonal antibody.
3. After an incubation period, a MAb-biotin specific to CPV vp2 protein is added and will bind to the protein if there is antigen in the sample.
4. To amplify the reaction, streptavidine-HRPO is added
5. This binding is detected by the development of a colorimetric reaction after the addition of the substrate, specific of HRPO.

APPLICATION

Detection of CPV (types 2, 2a & 2b) vp2 in faecal samples.

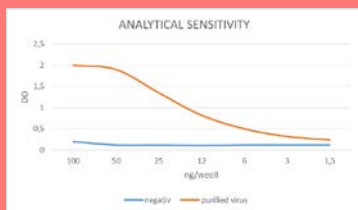
INTERPRETATION OF THE RESULTS

One cut off is used for the results interpretation. Samples will be **Positive** if their OD value is higher than the cut off. Samples will be **Negative** if their OD value is lower than the cut off.

VALIDACIÓN

1. Analytical sensitivity.


In order to determine the analytical sensitivity CPV purified viral antigen and negative antigen were titrated. The results indicated that the assay is able to detect 3ng of purified antigen.

2. Correlation with Haemagglutination (HA)

a. Experimental studies

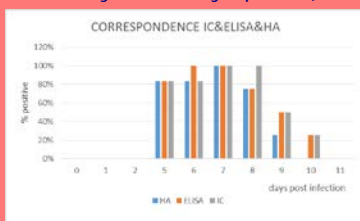Samples of 6 not vaccinated CPV infected dogs. 12 extractions were taken from day 0 p.i. the correlation between both assays was 93% being more sensitive INGEZIM PARVO CANINO DAS (Positive samples were confirmed by clinical symptoms)

b. Field studies

27 faeces samples of dogs, clinically suspicious of parvovirus and previously classified by other commercial ELISA, were analysed. The correlation with HA was 100%

		ELISA	
		POS	NEG
HA	POS	15	0
	NEG	0	12

COMPOSITION OF THE KIT

- 96 wells microtitration plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with conjugate
- Bottle with washing solution
- Bottle with diluent
- Bottle with substrate
- Bottle with stop solution

PRODUCT MANUFACTURED BY INGENASA

IT-73840 IT-73780 ISO 14001:2015 9191.INGE ISO 9001:2015 9175.ING2

Shelf life: **18 months**
Store at: 2°C-8°C