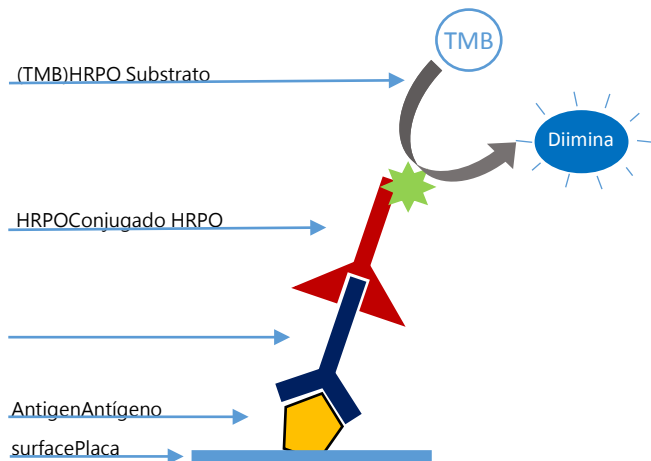
## INgezim EHRLICHIA

R.15.EHR.K1

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### REFERENCIAS:

*Development of a sensitive and specific indirect enzyme-linked immunosorbent assay based on a baculovirus recombinant antigen for detection of specific antibodies against Ehrlichia canis. Lissett López et al. J Vet Diagn Invest 19:635–642 (2007)*

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
## INgezim EHRLICHIA

R.15.EHR.K1

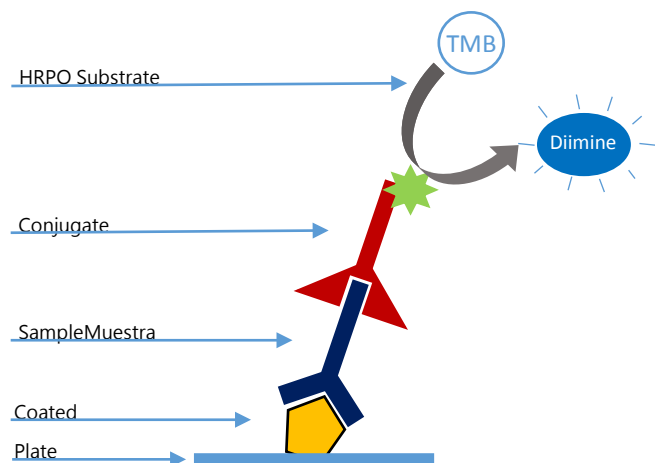**INgezim EHRLICHIA** is based on an indirect enzymatic immunoassay, which uses a monoclonal antibody (MAb), specific to canine immunoglobulins (Igs) and an extract of *E. canis* as antigen.

### TECHNICAL BASIS OF THE KIT

1. Plates are coated with a recombinant antigen of *Ehrlichia canis*. Sera samples are added and incubated.
2. If the samples contain specific antibodies to *Ehrlichia*, they will bind to the antigen.
3. When a MAb-PO specific of canine immunoglobulins is added, it will bind to the Igs previously bound to the antigen. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

### APPLICATION

Detection and/or titration of specific antibodies to *Ehrlichia* in canine sera samples.

### INTERPRETATION OF THE RESULTS

Two cut offs in relation to an index (IP) are established for the results interpretation:

**Screening:** IP values higher than the positive cut off are considered **Positive**; IP values lower than the negative cut off are considered **Negative**. Samples with IP values in the range of both cut offs will be considered **Doubtful**.

**Titration:** Since the IP is proportional to the level of antibodies, the titre of samples can be determined in correlation to this index.

### SENSITIVITY AND SPECIFICITY

#### Correlation with the reference technique (IFI):

##### 1. Study 1

The ELISA was initially developed by using a panel of 32 negative and 16 positive serum samples previously tested by IFA as the reference technique. The ELISA yielded 100% sensitivity and specificity on this initial panel.

##### 2. Study 2

A total of 98 positive sera from dogs naturally infected with *E. canis* and 157 negative serum samples were tested. Of the 255 sera analyzed, 226 yielded identical qualitative results (positive or negative) in both tests (indirect ELISA and IFA), corresponding to an accuracy value of 88.62%.

### REFERENCES

Development of a sensitive and specific indirect enzyme-linked immunosorbent assay based on a baculovirus recombinant antigen for detection of specific antibodies against *Ehrlichia canis*. Lissett López et al. *J Vet Diagn Invest* 19:635–642 (2007)

### COMPOSITION OF THE KIT

- 96 wells microtitration plates
- Vials with positive control
- Vials with Cut off control
- Vials with conjugate
- Bottle with washing solution
- Bottle with diluent
- Bottle with substrate
- Bottle with stop solution

PRODUCT MANUFACTURED BY INGENASA

Shelf life: **18 months**  
Store at: 2°C-8°C

Ed. 020217