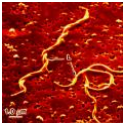
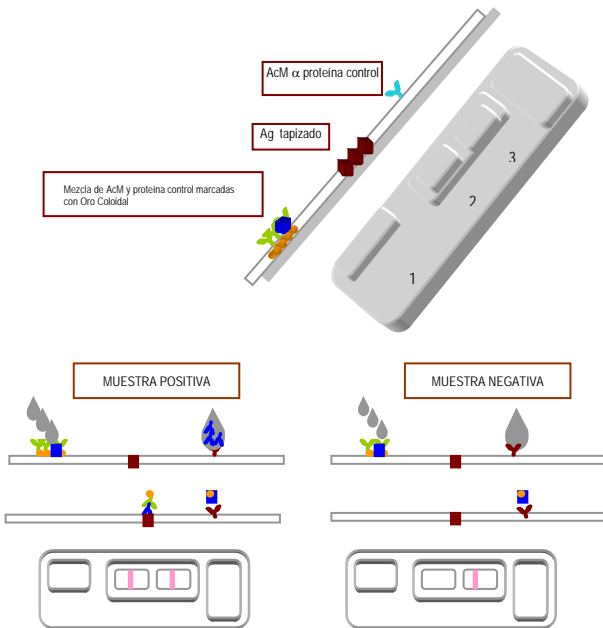
INGEZIM LYME-CROM (Ab)

R.15.LY.K41

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
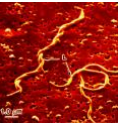
INGEZIM LYME-CROM (Ab)

R.15.LY.K41

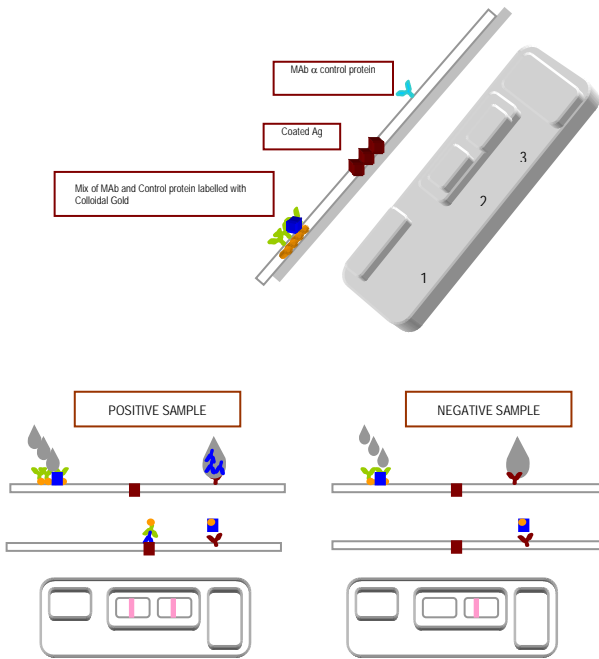Ingezim LYME-CROM is based on an indirect immunocromatography assay technique, which uses specific antibodies for canine immunoglobulins, and a recombinant antigen of *B.burgdorferi*.

TECHNICAL BASIS OF THE KIT

The device consists of a plastic casing with three windows:

- **Window1:** Where the Monoclonal Antibody specific of canine IgG and the Control Protein are conjugated to colloidal gold particles.
- **Window2:** Where the inactivated antigen of *B. burgdorferi* is adsorbed.
- **Window 3:** Where the MAb specific to the control protein is attached to indicate the validation of the assay.

APPLICATION

Detection of specific antibodies to *B. burgdorferi* in canine sera or plasma samples.

INTERPRETATION OF THE RESULTS

Positive sample: Red/pink lines in the Test zone and in the Control zone. **Negative sample:** Only a red/pink line in the Control zone. **Not valid assay:** No line in the Control zone

SENSITIVITY AND SPECIFICITY

Correlation with IFA technique

105 samples from dogs previously classified by IFI were analyzed by INGEZIM LYMECROM. The results obtained indicated that the correlation between both techniques was **96.1%**.

COMPOSITION OF THE KIT

- Diagnostic devices individually wrapped
- Diluent tubes, to prepare the dilution of the sera samples
- Dropper bottle dilution buffer
- Disposable tips

PRODUCT DISTRIBUIDO BY INGENASA

SHELF LIFE: **24 months**
Stored: 4°C-25°C

Ed. 020217