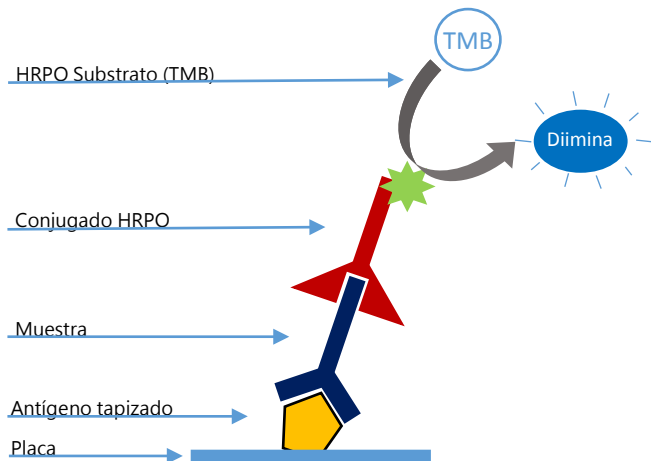
INgezim MIXOMATOSIS

R.17.MIX.K1

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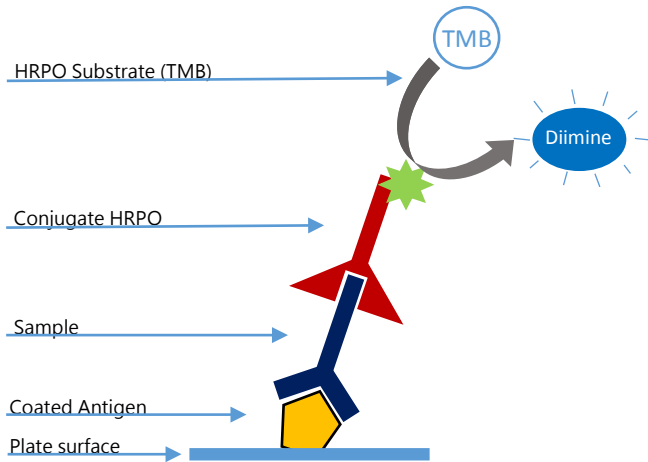
INgezim MIXOMATOSIS

R.17.MIX.K1

INgezim MIXOMATOSIS is an immunoenzymatic assay based on an indirect ELISA technique to detect antibodies specific to mixomatosis virus.

TECHNICAL BASIS OF THE KIT

1. Plates are coated with antigen of mixomatosis virus. Serum samples are added to the wells and incubated.
2. If the sample has antibodies specific of the virus, they will bind to the antigen.
3. When the conjugate specific of rabbit immunoglobulins is added, it will bind to the specific antibodies present in the sample. This binding is detected by adding a specific substrate.

APPLICATION

Detection of specific antibodies to Mixomatosis virus.

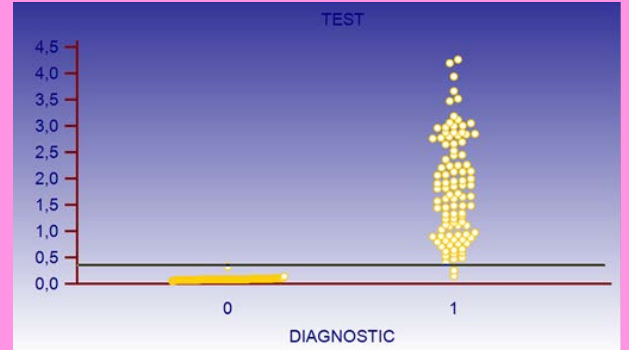
INTERPRETATION OF THE RESULTS

The assay establishes two cut offs. Samples will be considered **Positive**, if their OD value is higher than the positive cut off; **Negative**, if their OD value is lower than the negative cut off. Samples with an OD between both values must be considered **doubtful**.

SENSITIVITY AND SPECIFICITY

Validation of the assay was made with 204 sera of animals previously classified by other techniques: 97 negative sera, 7 doubtful, 50 weak positive and 50 strong positive.

Results obtained indicated 98% sensitivity and a specificity higher than 99%.

COMPOSITION OF THE KIT

- Microtitration plates of 96 well
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate

PRODUCT MANUFACTURED BY INGENASA
SPANISH REGISTRATION NUMBER: 3367 RD

SHELF LIFE: 18 months
Stored at 2°C-8°C