
## INGezim TUBERCULOSIS DR

R.10.TB.K0

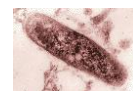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


## INGezim TUBERCULOSIS DR

R.10.TB.K0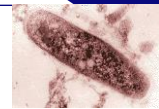

**INGezim TUBERCULOSIS DR** is an immunoenzymatic assay which uses MPB83 recombinant protein of *Mycobacterium bovis*, both, to coat the plate and as conjugate.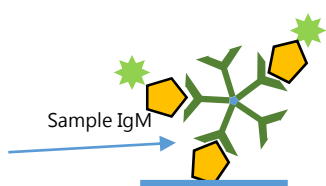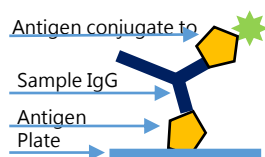

### TECHNICAL BASIS OF THE KIT

- Plates are coated with *M. bovis* MPB83 recombinant protein. Serum samples are added to the wells and incubated.
- If the sample contains specific antibodies to *M. bovis*, they will bind to the antigen.
- When a HRP conjugated recombinant MPB83 protein is added, it will bind to the antibodies present in the sample if the serum sample is positive. This binding is detected by adding a specific substrate.

### APPLICATION

Detection of specific antibodies to MPB83 of *M. bovis*. The assay is valid for use with wildlife (terrestrial and aquatic mammals and birds). Do not use with cattle. Serum, plasma and blood in paper samples can be used individually. Small ruminants (goats and sheep) milk samples (individual and tank) can be used

### INTERPRETATION OF THE RESULTS

The assay establishes 2 cut offs. Samples will be considered **Positive**, if its OD value is higher than the positive cut off; **Negative**, if its OD value is lower than the negative cut off. **Doubtful** in case that the OD value is within the range of both cut offs.


## DIAGNOSTIC SENSITIVITY AND SPECIFICITY

### SERUM

The assay has been evaluated with deer (139), fallow deer (365), alpaca (451), badgers (61), wild boar (351), Nebrodi black pigs (106), Iberian pigs (183), goats (616) and exotic animals (terrestrial and aquatic mammals and birds) (170). Different techniques has been used as gold standard depending on the study.

#### 1. Sensitivity

This assay shows sensitivity values within the range of 53 to 98% depending on the specie studied and the gold standard.

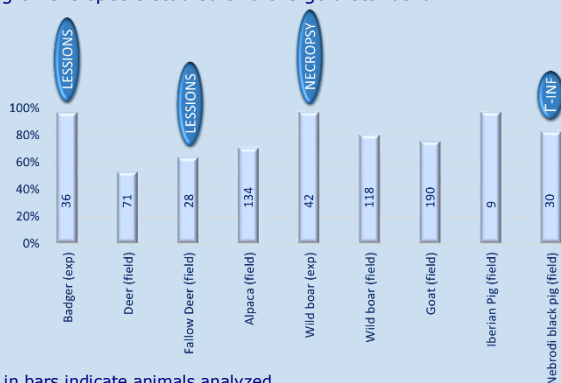\*Numbers in bars indicate animals analyzed

#### 2. Specificity

This assay shows specificity values within the range of 92 to 100% depending on the specie studied and the gold standard.

\*Numbers in bars indicate animals analyzed

### ANALYTICAL SPECIFICITY

The assay has been evaluated with 40 sera positive to Paratuberculosis and to Tuberculosis, 2 positive and 2 animals negative to both diseases. Results obtained indicated that **there is not cross-reaction with antibodies specific of paratuberculosis.**

### MILK of small ruminants

The performance of the assay using milk samples was determined using 25 milk tank samples of goats previously classified by lesions and 184 milk tank samples of sheep from herd with low prevalence. Results obtained indicated a good correlation regarding the expected ones.

### BLOOD SAMPLES IN FILTER PAPER

The assay has been evaluated using 305 wild boar blood in filter samples previously catalogued by PCR (288 negative, 5 PCR positive to *M. microti*, 1 PCR positive to *M. avium hominisuis*, 9 PCR positive to *M. bovis*, 2 positive without specie identification). Results obtained indicated 89% sensitivity (8/9) and 99% specificity. These values were similar to the obtained with wild boar serum samples in other studies.

### COMPOSITION OF THE KIT

- Microtitration plates of 96 well
- Vials with Positive Control
- Vials with Negative Control
- Vials with Peroxidase Conjugate
- Bottle with Washing Solution
- Bottle with diluent
- Bottle with stop solution
- Bottle with substrate (TMB) ready to use

PRODUCT MANUFACTURED BY INGENASA  
REGISTERED IN SPAIN (10012 RD)

SELF LIFE: **12 months**  
Stored 2°C-8°C