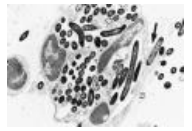
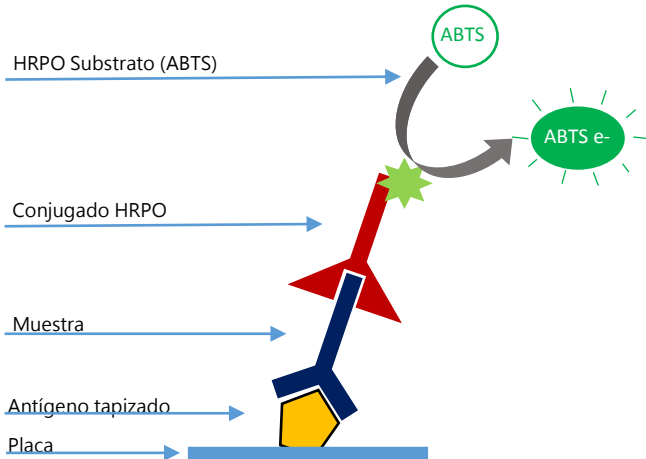
## INGEZIM MAL ROJO

R.11.MR.K1

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

## INGEZIM MAL ROJO

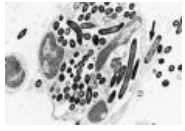R.11.MR.K1

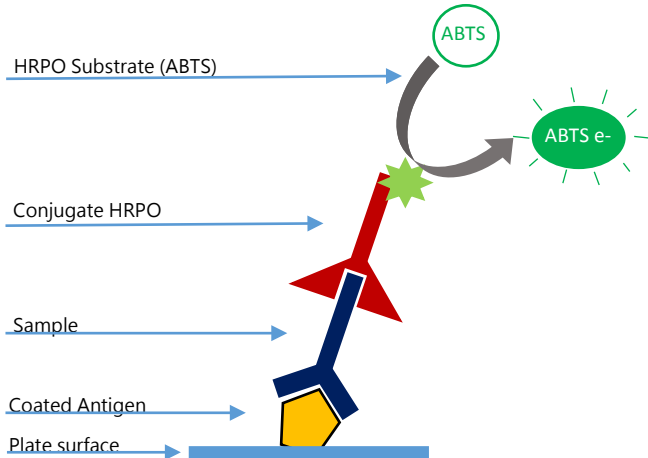**INGEZIM MAL ROJO** is based on an indirect ELISA technique, which uses a monoclonal antibody (MAb) specific to porcine immunoglobulines (Igs).

### TECHNICAL BASIS OF THE KIT

1. Plates are coated with a soluble extract of *E. rhusiopathiae* antigen. Serum samples are added and incubated.
2. If the samples contain specific antibodies to *E. rhusiopathiae*, they will bind to the antigen.
3. When the MAb-PO specific of porcine IgGs is added, it will bind to the IgGs bound to the antigen. The binding is detected by the development of a colorimetric reaction after the addition of the substrate.

### APPLICATION

Detection and /or titration of specific antibodies to *Erysipelothrix rhusiopathiae* in porcine sera samples

### INTERPRETATION OF THE RESULTS

The samples will be considered as **Positive** when the OD value is equal to or higher than the positive cut off. The samples will be considered as **Negative** when the OD value is equal to or lower than the negative cut off. Titre of the sample will be the last dilution which OD value is higher than the cut off.

### SENSITIVITY AND SPECIFICITY

#### 1. Correspondence with the reference technique (slow agglutination).

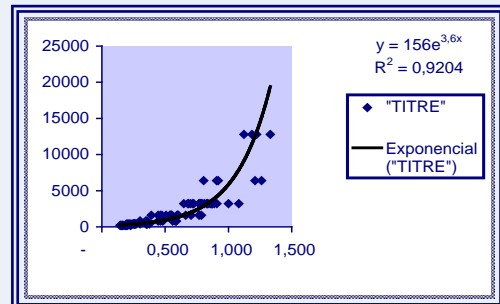
255 sera were analyzed, to determine the correspondence between INgezim MR and the slow agglutination technique.

The results obtained indicated a correspondence of **93%**

### ONE WELL TITRATION OF SERA

It is possible to obtain an approximate titre of the sera based on its OD value, by using a formula which corresponds with an exponential curve.

#### ONE WELL TITRATION FOR 11MRK1

### COMPOSITION OF THE KIT

- Microtitration plates of 96 wells divided into 12 strips of 8 wells each.
- Vial with Positive Control ready to use
- Vial with Negative Control ready to use
- Vial with Peroxidase Conjugate 100x concentrated
- Bottle with Washing Solution concentrated
- Bottle with diluent 5x concentrated (DE01-05)
- Bottle with substrate ABTS
- Bottle with stop solution

PRODUCT MANUFACTURED BY INGENASA

SHELF LIFE: **18 months**  
Stored at 2°C-8°C

Ed.020217