INGEZIM CRYPTOCROM

Prod Ref: 10.CRY.K4

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
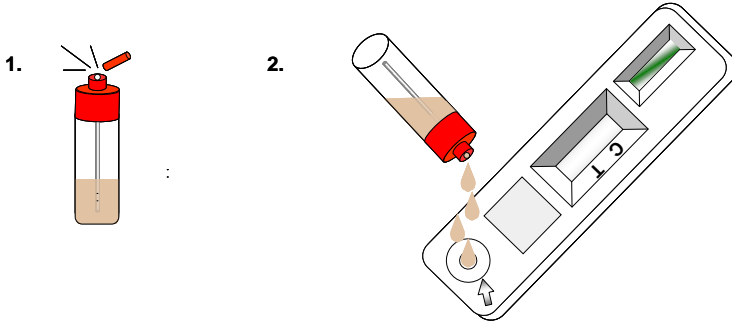
Esperar 10 minutos e interpretar:

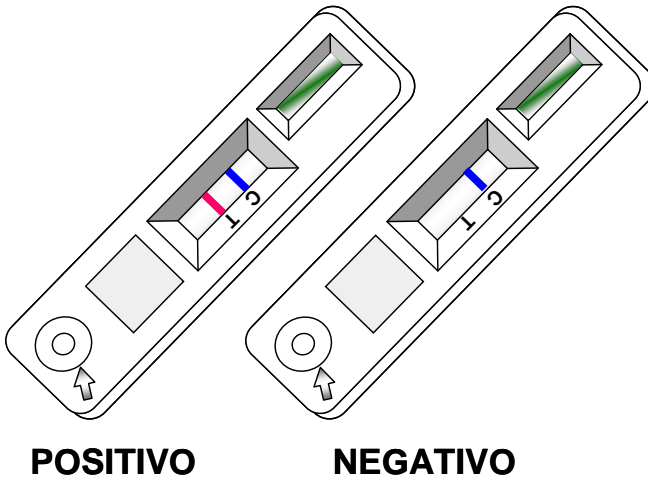I. TECHNICAL BASIS

The technical basis of this test is a direct immunochromatography, where we are using a mixture of monoclonal antibodies specific to *Cryptosporidium parvum*, which are able to detect all the shapes of the vital cycle of the parasites.

II. STORAGE OF THE REAGENTS

All the reagents must be kept between 4 and 25 °C.

III. PRECAUTIONS

1. Read carefully the instructions of use.
2. **Kept the reagents at room temperature before their use.**
3. Do not mix reagents or instruction from different kits.
4. Avoid any contamination of the reagents.
5. Do not use the kits after the expiry date.
6. There should be no eating, drinking or smoking where specimens or kit reagents are being handled.

IV. TEST PROCEDURE

1. Preparation of samples:

Unscrew the cap of the collection tube containing the diluent. Introduce the applicator stick into the faeces and collect a small amount of them. Reinsert the applicator stick into the tube and screw the cap. For liquid or semisolid stools add 100 µl of stool using an appropriate pipette. Shake the collection tube to assure a right sample dilution.

2. Sample addition:

Open the aluminium bag just in the moment to realise the assay. Remove the test device from the bag and place it on a flat surface. Break the top part of the tube and dispense 5 drops into the round window. Wait 10 minutes and read.

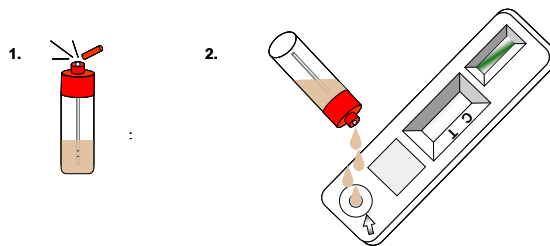
3. Interpretation of results:

In order to be sure that the chromatographic assay has run properly, a blue line must appear in control area (C).


- If appears a red/pink line in the test area (T) the sample assayed must be reported as POSITIVE to Cryptosporidium.
- If there is no line in the test area (T) and there is a blue line in the control area (C) the sample must be reported as negative to Cryptosporidium.
- NO VALID TEST: If there is no blue line into the control area (C) the assay can't be considered valid.

No result must be considered after 10 minutes.

Add 5 drops of diluted sample into the round window.

Wait 10 minutes and read the results:

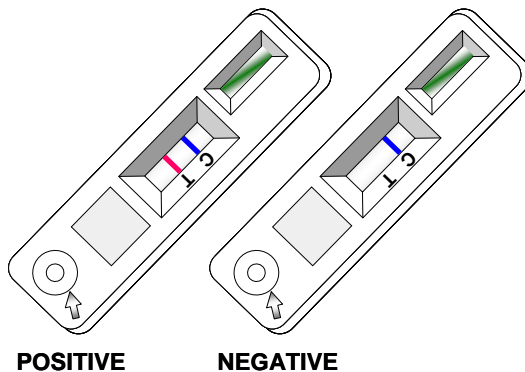Developed and manufactured by:

Distributed by:

INMUNOLOGIA Y GENETICA
APLICADA, S.A.
C/ Hnos. García Noblejas, 39
28037- MADRID (SPAIN)
Tlf: +34 91368.05.01/04
Fax: +34 91 408.75.98
E-mail: ingenasa@ingenasa.com
www.ingenasa.es

IT-73840 ISO 14001:2004 ISO 9001:2008
IT-73780 9191.INGE 9175.ING2