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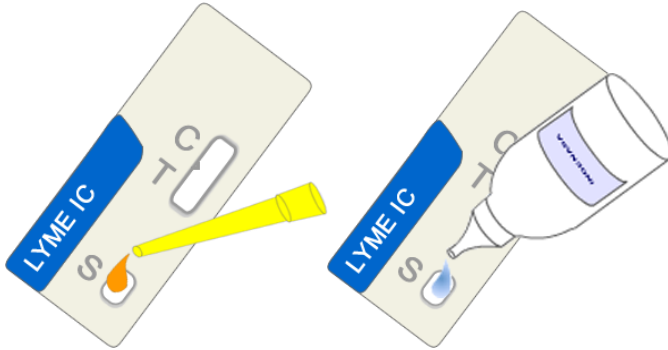
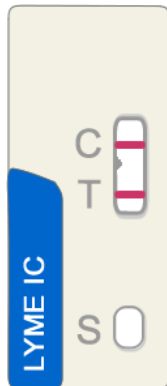
NEGATIVO

I. TECHNICAL BASIS

Ingezim Lyme-crom is a rapid immunochromatographic assay designed for the detection of antibodies to *Borrelia burgdorferi* using a recombinant protein immobilised to the solid phase membrane..

II. STORAGE:

All the reagents must be kept between 18 and 25 °C.

IMPORTANT! DO NOT FREEZE

III. PRECAUTIONS:

1. Read carefully the instructions of use.
2. **Kept the reagents at room temperature before their use.**
3. Do not mix reagents or instruction from different kits.
4. Avoid any contamination of the reagents.
5. Do not use the kits after the expiry date.
6. There should be no eating, drinking or smoking where specimens or kit reagents are being handled.
7. Do not pipette by month.

IV. TEST PROCEDURE:

1. Preparing the test device:

Remove the test device from the protective pouch and place it on a flat surface.

NOTE : Do not remove the device from the pouch until ready to use.

2. Sample addition:

Gently mix the sample by inverting. Place 1 tip on the precision pipette and aspirate 10 µl of sample (serum, plasma or whole blood). Depressing bottom of the pipette to dispense the sample into window "S" (sample). Make sure as much of the sample is applied as possible.

Serum or plasma: Fresh, or stored at 2°-7° C, may be used in this test. Serum or plasma may be stored for up to 5 days at 2°-7° C. For longer storage, samples should be frozen.

Whole blood: Must be anticoagulated (EDTA, heparin, etc...) and may be used either fresh or after refrigeration for up to 5 days. It is important that

the sample does not be left in the pipette tip too long as clotting may occur. In this case it is important to have the device ready in advance.

Previously frozen or older samples must be centrifuged before use.

PRECAUTION: *Never touch the nitrocellulose membrane with the pipette.*

3. Solution buffer addition:

Holding the diluent bottle vertically, promptly add 2 drops of the buffer to the window "S".

4. Read and interpretation:

After 10-15 minutes read the results.

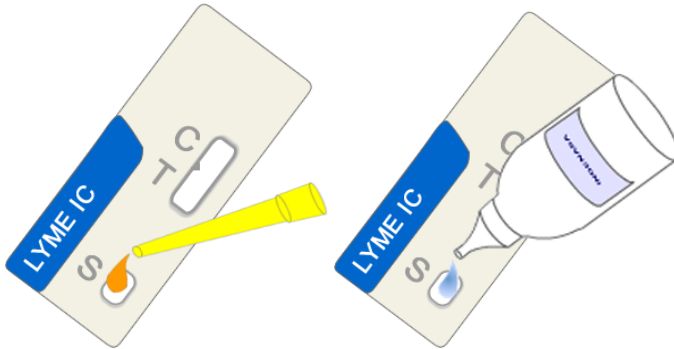
POSITIVE RESULTS: the test is positive if two coloured bands appear. One band will appear in the window "T" (Test line) and one in the window "C" (Control line).

NEGATIVE RESULT: There is no line in the window "T" (Test line) and there is a purple coloured line in the window "C" (control line).

NO VALID TEST: If there is no line into the window "C" the assay can't be considered valid and should be repeated.

Coloured bands, which appears after 15 minutes are not diagnostic, and should be ignored.

Add 10 µl of sample and 2 drops of buffer solution into the window "S"

Wait 10-15 minutes and read:

POSITIVE

NEGATIVE

Developed and manufactured in
Spain by:

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S.A.